# Polaroid

IT

EN

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## HEARING AID

EN

**The product:** Polaroid BTE and ITE hearing aids increase and improve your auditory functions. They are intended for individuals with mild to medium hearing loss, to be used as occasional hearing aid. They can be used when needed: during a movie, a sporting event, in a meeting or in during a lecture or on any occasion in which your auditory function needs a little help.

### HOW TO USE:

- Carefully read the operating instructions manual and familiarise yourself with the device and on how to use it properly.
- The first few times we suggest to use the unit in quiet environments; do not use it immediately in noisy environments.  
Take your time to get used to the sound of your new device, and increase the usage time gradually.

MALFUNCTIONS	CAUSE	POSSIBLE SOLUTIONS
Device is not working, you can't hear any sound.	Battery is not inserted properly	Insert the battery ensuring correct polarity
	Battery level is low	Replace it with a new battery
	Battery is not connected properly	Clean the contacts or raise them with tweezers
	Headset damaged or presence of earwax	Clean or replace it with a new one
Volume is low	Headset damaged or presence of earwax	Clean or replace it with a new one
	Hearing drop, loss	Contact your professional hearing specialist
The sound is intermittent	Possible humidity	Open the battery compartment and leave it open 24 hours to let the moisture evaporate.
	Unit is defect	Call our service center
You can hear a whistle (Larsen effect)	Fault	Power cycle unit
	Volume is too high	Reduce the volume
	Earbuds size is wrong	Find proper fitting earbuds. Keep in mind that they need to cover the majority of the ear canal. Turn it on and increase the volume gradually.

## Warnings

-  The sole purpose of the hearing aid is to amplify and transmit sound, offsetting in this way hearing loss. Hearing instruments should be used only by the intended user. Other users are not allowed because the hearing aids can potentially cause hearing damage.
-  The hearing aid must be used only according to the instruction manual.
-  The hearing aid will not restore normal hearing conditions and does not prevent or fix hearing loss due to physical conditions.
-  Do not use hearing aids in explosion hazardous areas.
-  Allergic reactions to hearing aids are rare. However, if you are experiencing itching, redness, soreness, inflammation or a burning sensation in or around the ear, consult a physician.
-  In the unlikely event that some parts remain in the ear canal after the removal of the hearing aid, consult a doctor immediately.

- 
-  Remove the hearing aid in case of CT, MRI and other examinations with electromagnetic equipment.
  -  Do not use any alcohol to clean your hearing aid and/or your custom earbuds.
  -  The small tubes and custom earbuds should never be rinsed or submerged in water, as water droplets may remain in the tube, blocking the sound or damaging the electric components of the hearing aids.
  -  Children must wear the hearing aid under the supervision of an adult. Hearing aid not suitable for children under three years old.
  -  Patients with mental disorders should consult a specialist before using the hearing aid.

EN

DISPOSE OF USED BATTERIES IN PROPER RECYCLING CONTAINERS

### **Repair:**

In case of failure contact only qualified and authorised personnel.

### **Do not open the device under any circumstances.**

The device cannot be repaired by non-qualified personnel and does not require internal maintenance and/or lubrication.

EN

### **Maintenance:**

When not in use, store the device in the proper case. Clean the equipment with a soft, dry cloth. Do not use abrasive or volatile cleaners, oil, solvents, gasoline, etc. Never immerse the appliance and the accessories in water. Store in a clean, dry place. Do not subject the device to extreme heat or cold, or moisture or direct sunlight. Remove the batteries if you do not use the device for three or more months.

## Reference standards:

The device meets the requirements of European directive 93/42/CEE.

Electromagnetic compatibility: The device conforms to the European standard conventions EN 60601-1-2. The device passed the EMC test relating to EN 60118-13:2011.

EN

**Appropriate disposal of this product** (electrical & electronic waste) (Valid in the European Union countries with recycling systems). The symbol on the product or on its documentation indicates that the product complies with the electrical and electronic waste regulations and It should not be disposed of in household rubbish. You are responsible for assigning the equipment at end of life in the appropriate collection facilities, failing which you would incur sanctions accordingly to the current waste legislation. For more information regarding the collection systems available please contact your local waste disposal service.

## Warranty

- The warranty is valid for 2 years from the date of purchase against any design faults or material failure.
- The warranty covers the replacement and/or free repair of defective components.
- The warranty does not cover the accessories supplied with the unit or parts subject to normal wear.
- The shipping costs are borne by the user.
- Repairs out of warranty are borne by the user.
- The warranty is void if the unit has been tampered with, if the defect was caused by improper use or if the damage is not attributable to the manufacturer (accidental fall, rough transportation, etc.).

EN

- The warranty does not involve any compensation for direct or indirect damage toward persons or property of any kind during the inefficiency period.
- The warranty is valid from the date of purchase certified by the possession of the receipt or the invoice.

EN

EN

## INDEX

EN

DIGITAL AIR 3D Model	p.102
DIGITAL SUPERIOR 3D Model	p.128
DIGITAL INVISIBLE 3D Model	p.154

EN

((( DIGITAL AIR • 3D

REF 101

Other codes:

**Digital PRO 3D**

REF 104

## Hearing aid description

The DigitalAir 3D Polaroid hearing aid is digital, with 4 signal processing channels, intelligent and adaptive noise reduction system, highly effective acoustic feedback suppression function.

The DigitalAir 3D Polaroid hearing aid helps the user not only to have a clean hearing in a quiet environment, but also in a noisy environment.

The product design meets the ergonomics with comfortable and powerful functions. the hearing aid offers the possibility to choose between two preset settings.

The description is the same for the **Digital PRO 3D Polaroid** hearing aid.

EN

**EN**

Technical Specifications	DIGITAL AIR-3D	
	Prog. 1	Prog. 2
MAX-OSPL 90	121.8 dB	119.5 dB
HFA OSPL 90	114.2 dB	111.5 dB
Peak Gain	31.9 dB	29.3 dB
HFA Full on Gain	25.5 dB	22.6 dB
Total Harmonic Distortion:		
500Hz (%)	0.1	0.2
800Hz (%)	0.2	0.1
1600Hz (%)	0.3	0.3
EQ Input Noise Level	24.2 dB	24.4 dB
Frequency Range	$\leq 250 \text{ Hz} \geq 6000 \text{ Hz}$	
Attack Time	13 ms	9 ms
Release Time	48 ms	38 ms
Current	1.64 mA	1.79 mA

## Structure Characteristics and Operating Principle

Hearing aid is consisted by the microphone, integrated circuit and receiver. The microphone can transform the received sound to electric signal, the integrated circuit will amplify the electric signal, the receiver will accept the amplified electric signal and restore to sound, to achieve the sound amplification purpose and make the compensation for the hearing loss user, to help the hearing loss user working and living like normal people.

EN

## Operating Condition

### Working Conditions:

Temperature: 0°C~+38°C, Humidity: 0%~80%, Voltage: DC1.4V

### Storage Conditions:

Temperature: -20°C~+50°C, Humidity: 0%~93%

## Impact on the Environment

Please dispose of the used battery according to local regulations to protect the environment.

## Safety

EN

- **Warning:** Keep small parts that can be swallowed out of children's reach.
- **Warning:** Hearing aids are able to provide more than 132 dB SPL.  
We suggest having the acoustic amplifier mounted by a professional operator
- **Warning:** Not to use the hearing aids in such areas with risk of explosive or oxygen enriched atmospheres.
- **Warning:** The hearing aid must only be used by tended person and not by others.  
For hearing aids with wireless transmission: warning to check first before using the hearing aids in areas where electronics or wireless devices are restricted;  
Statement required about the special needs of particular users.  
e.g. small children or mentally disabled persons.
- **Warning** about common conditions that:  
**Danger:** Take out battery if there is long time non-use of hearing aids.  
**Danger:** Do not charge the battery or throw the used batteries into fire to avoid any explosion.

**Danger:** Do not mix the battery with food, medicines; Do not put the battery into the mouth. If a battery is swallowed, please consult a doctor immediately. It is life-threatening to swallow the hearing aid battery.

The hearing aid should be only used by the hearing impaired, not by any other.

- **Special User:**

- (1) Children can't wear the hearing aid without the guidance of adult.
- (2) Mental patients should consult a physician before wearing a hearing aid.
- (3) The hearing aid is not applicable for children under age of 3 years.

EN

## Troubleshooting Guide

EN

FAILURE	CAUSE	SOLUTION
Whistling	Hearing aid is not worn correctly	Wear it again and correctly
	Too much earwax	Clean earwax
No Sound	Power off	Power on
	Battery is not installed	Install the battery
	Low battery or no power	Change a new battery
	Sound tube is blocked	Clean the sound tube
	Ear mold or ear dome is blocked	Clean the ear mold or ear dome
	Hearing aid is in sleep mode	Long press the button 5 seconds to awaken the hearing aid

FAILURE	CAUSE	SOLUTION
Quiet Sound	Low volume	Increase the volume
	Low battery	Change a new battery
	Hearing aid is damped	Dry the hearing aid
	Sound tube is blocked	Clean the sound tube
	Ear mold or ear dome is blocked	Clean the ear mold or ear dome
High Power Consumption	Hearing aid is still working when not using	Power off when not using
	Hearing aid is damped	Dry the hearing aid
Intermittent Sound	Battery failure	Change a new battery
	Sound tube is blocked	Clean the sound tube
	Ear mold or ear dome is blocked	Clean the ear mold or ear dome

EN

If the problem is not solved by applying the procedures above, please contact the store where you purchased it or contact Jordan TE.CH customer service

## Precautions For Hearing Aid

### a) Moisture Prevention

Any moisture can have a negative impact on the hearing aid. Moisture prevention is helpful to extend the lifetime of hearing aid. So please remember not to use or store hearing aid in humid condition. Please dry the hearing aid periodically.

### b) Water Prevention

Keep hearing aid away from water. Please take off the hearing aid in advance and keep properly when swimming, shower, washing the head or face. If hearing aid falls into the water accidentally, do not use any dryer to dry it. You can use the soft and clean cotton cloth to dry it and put it in a ventilated and dry environment. If any failure occurs, please contact the hearing center where you acquired the hearing aid or customer service center of Jordan TE.CH. SA company.

### c) High Temperature Prevention

Never expose the hearing aid to high-temperature or prolonged sunlight.

#### d) Dropping Prevention

Hearing aid is precise electronic device. Please prevent it from dropping or violent vibration and shock.

EN

#### e) Others

- Do not wear your hearing aids during short-wave diathermy, X-ray, MR scans, CT scans or similar radiation treatments.
- Please use professional battery for hearing aid.
- If you not use hearing aid for long time, please take out the battery to avoid battery leakage.
- Battery should be put in a safe place which is out of baby or children's reach. If any swallowing accident happens, please consult a doctor immediately.

## Hearing Rehabilitation Plan

Every person reacts differently the first time he or she wears hearing aids. The familiarization phase therefore differs from person to person. Below is a rehabilitation plan for first time users or those using new hearing aids.

### a) The First Stage (1-3 weeks)

1 to 2 hours wearing per day, you are suggested to wear in a relatively quiet environment to familiarize and distinguish various sounds.

### b) The Second Stage (1-3 weeks)

4 to 8 hours wearing per day, you may wear your hearing aids and walk in a relatively quiet environment to listen around.

**c) The Third Stage (1-3 weeks)**

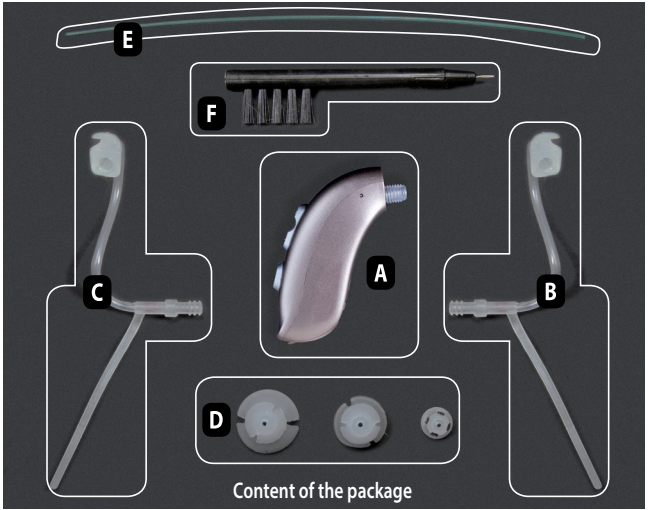
You may wear hearing aids and go to public areas. You may have dialogue with your friends or even strangers.

**d) The Fourth Stage (1-3 weeks)**

You may prepare for travelling to another city, go to supermarket, restaurant, participate in music concert or watch video.

EN

Content of the package

The unit can be worn on either ear, using the respective BTE tubes.

The kit consists of.



- A** Hearing Aid
- B** Right BTE tube
- C** Left BTE tube
- D** Earbuds in different sizes, S-M-L.
- E** Rigid wire for BTE tubes cleaning.
- F** Ear cleaning brush.

REF 150

**BATTERIES  
NOT INCLUDED.**


**Compatible Batteries:  
ZINC-AIR,  
model 13**

**EN**

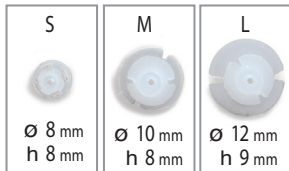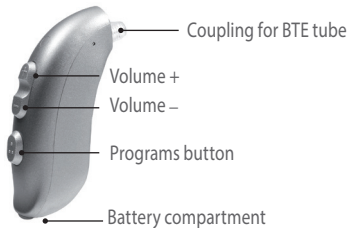Select the BTE tube:  
the right one, **RED B** or the left, **BLUE C** depending on which ear the hearing aid will be applied on.

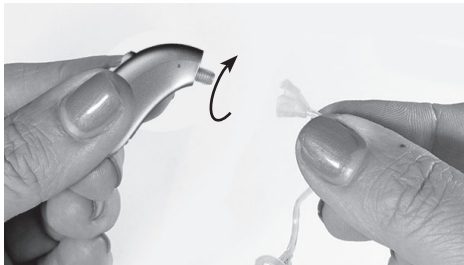
EN

Apply the tube to the device: the end of the tube is to be tightened clockwise in the threaded plug, exerting pressure.


Choose the most suitable earbud **D** between the three different sizes available (SML). If this is your first time using the device, you may want to try M size earbuds first. Assemble by pressing the earbud **D** on the BTE tube.

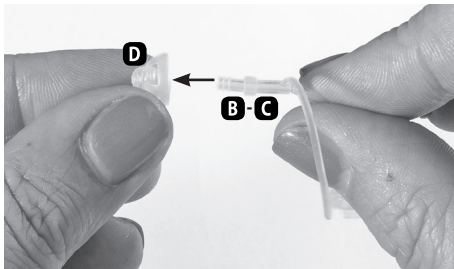EN

Open the battery compartment door.  
Do not push hard, so as not to damage  
the compartment.

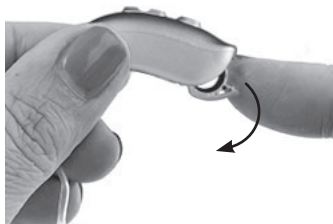EN


### Compatible Batteries:

**ZINC-AIR,  
model 13.**

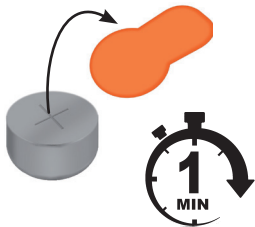
If you are using zinc-air batteries:  
remove the plastic battery and wait  
about 1 minute before inserting it  
back into the compartment.

Insert the battery into the compartment ensuring correct polarity.

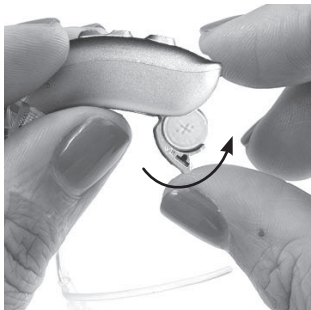Close the battery compartment gently. If the battery is not properly positioned the battery compartment cannot be closed. The device is now switched on.


EN

At first use, turn the volume to a minimum by pushing the volume button (see figure): Press the bottom to decrease it. Verify the functioning of the device by placing it between two hands closed: if the device is working properly it will produce a sound/whistle (Larsen effect).

EN

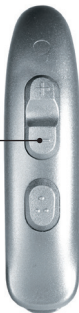Press to decrease  
the volume


## Installing the hearing aid:

1. Make sure you have properly inserted the battery and you have closed the compartment.
2. Pull up on the top of your ear to help the insertion of the earbuds into the ear canal and the setting of the tube flap into the ear.
3. Hold the headset with your thumb and index finger on the outside, next to the tube.
4. Gently insert, slightly tilted, the tip of the earbud in the ear canal.
5. Turn hearing aid rearwards.
6. Gently push the earbud with the fingertips until it is properly seated.
7. Place the hearing aid behind the ear.

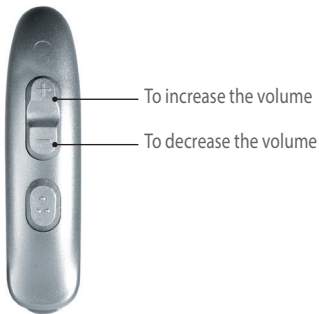EN

You may hear a slightly annoying sound that will disappear when the device is properly positioned within the ear canal. If the positioning is correct and the sound is still perceptible, try with a different size and repeat the operation. Slowly adjust the volume as needed. The Hearing aid has seven different volume intensities marked each one by a "beep". The end of the scale, both for minimum and maximum volume, is marked by two "beeps".

How to select  
**amplification programs.**

Prog.	Beeps	Operation
1	1 	silent environments
2	2  	noisy environments

EN

Press for the  
program selection

The prominence highlighted with the arrow is the stabilizer of the tube and must be inserted into the earcup.

EN

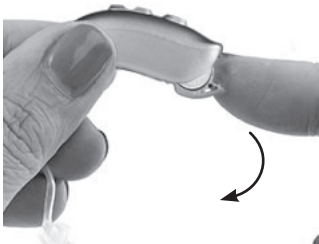
Once the headset is inserted into the ear canal, place the stabilizer inside the earcup to complete the assembly.

At the end of use, remove the device.

To turn off the unit open the battery compartment door.

EN

## Maintenance

When using hearing aids, the earwax accumulates in the ear canal and in the tube. The ear wax can affect the quality of the hearing aid. Please clean the ear canal and the hearing aid regularly.

EN

### a) Change the earbuds

- Remove the earbuds from the hearing aid  
Hold the device with your left hand and the tube with your right hand to separate them from each other.
- Install the earbud

### b) Replace the tube

- Remove the tube  
Hold the triangular base of the tube and turn it counterclockwise approximately 45 °.  
Remove the tube from the device.
- Assemble the new tube.

c) Clean the hearing aid

- \* Clean the hearing aid with a soft, dry cloth.

- \* Clean the hearing aid on a soft ground like the desk with a soft towel to avoid damage when falling down.

EN

**Warning:** never use any liquid to clean the hearing aid

d) Humidity prevention

Place the hearing aid in the box to prevent moisture and prolong the hearing aid.

e) Impact Prevention

Please place the hearing aid in the box or when not in use.

EN

((( DIGITAL SUPERIOR • 3D

REF 100

## Hearing aid description

The Digital Superior 3D Polaroid hearing aid is digital, with 4 signal processing channels, intelligent and adaptive noise reduction system, highly effective acoustic feedback suppression function.

The Digital Superior 3D Polaroid hearing aid helps the user not only to have a clean hearing in a quiet environment, but also in a noisy environment.

The product design meets the ergonomics with comfortable and powerful functions.

Unlike the DigitalAir 3D, the superior 3D offers a single preset use setting.

EN

**EN**

Technical Specification	DIGITAL SUPERIOR-3D
Parameters	Values
MAX-OSPL90 (dB)	110.8
HFA-OSPL90 (dB)	104.4
Peak Gain (dB)	36
HFA FULL-ON GAIN (dB)	29
Frequency Range	$\leq 200\text{Hz}$ $\geq 6600\text{Hz}$
THD at 500Hz	0.4%
THD at 800Hz	0.3%
THD at 1600Hz	0.5%
Eq. Input Noise Level (dB)	25
Current (mA)	1.35
Attack Time (ms)	10
Release Time (ms)	77

## Structure Characteristics and Operating Principle

Hearing aid is consisted by the microphone, integrated circuit and receiver. The microphone can transform the received sound to electric signal, the integrated circuit will amplify the electric signal, the receiver will accept the amplified electric signal and restore to sound, to achieve the sound amplification purpose and make the compensation for the hearing loss user, to help the hearing loss user working and living like normal people.

EN

## Operating Condition

### Working Conditions:

Temperature: 0°C~+38°C, Humidity: 0%~80%, Voltage: DC1.4V

### Storage Conditions:

Temperature: -20°C~+50°C, Humidity: 0%~93%

## Impact on the Environment

Please dispose of the used battery according to local regulations to protect the environment.

## Safety

EN

- **Warning:** Keep small parts that can be swallowed out of children's reach.
- **Warning:** Hearing aids are able to provide more than 132 dB SPL.  
We suggest having the acoustic amplifier mounted by a professional operator
- **Warning:** Not to use the hearing aids in such areas with risk of explosive or oxygen enriched atmospheres.
- **Warning:** The hearing aid must only be used by tended person and not by others.  
For hearing aids with wireless transmission: warning to check first before using the hearing aids in areas where electronics or wireless devices are restricted;  
Statement required about the special needs of particular users.  
e.g. small children or mentally disabled persons.
- **Warning** about common conditions that:  
**Danger:** Take out battery if there is long time non-use of hearing aids.  
**Danger:** Do not charge the battery or throw the used batteries into fire to avoid any explosion.

**Danger:** Do not mix the battery with food, medicines; Do not put the battery into the mouth. If a battery is swallowed, please consult a doctor immediately. It is life-threatening to swallow the hearing aid battery.

The hearing aid should be only used by the hearing impaired, not by any other.

- **Special User:**

- (1) Children can't wear the hearing aid without the guidance of adult.
- (2) Mental patients should consult a physician before wearing a hearing aid.
- (3) The hearing aid is not applicable for children under age of 3 years.

EN

## Troubleshooting Guide

EN

FAILURE	CAUSE	SOLUTION
Whistling	Hearing aid is not worn correctly	Wear it again and correctly
	Too much earwax	Clean earwax
No Sound	Power off	Power on
	Battery is not installed	Install the battery
	Low battery or no power	Change a new battery
	Sound tube is blocked	Clean the sound tube
	Ear mold or ear dome is blocked	Clean the ear mold or ear dome
	Hearing aid is in sleep mode	Long press the button 5 seconds to awaken the hearing aid

FAILURE	CAUSE	SOLUTION
Quiet Sound	Low volume	Increase the volume
	Low battery	Change a new battery
	Hearing aid is damped	Dry the hearing aid
	Sound tube is blocked	Clean the sound tube
	Ear mold or ear dome is blocked	Clean the ear mold or ear dome
High Power Consumption	Hearing aid is still working when not using	Power off when not using
	Hearing aid is damped	Dry the hearing aid
Intermittent Sound	Battery failure	Change a new battery
	Sound tube is blocked	Clean the sound tube
	Ear mold or ear dome is blocked	Clean the ear mold or ear dome

EN

If the problem is not solved by applying the procedures above, please contact the store where you purchased it or contact Jordan TE.CH customer service

## Precautions For Hearing Aid

### a) Moisture Prevention

Any moisture can have a negative impact on the hearing aid. Moisture prevention is helpful to extend the lifetime of hearing aid. So please remember not to use or store hearing aid in humid condition. Please dry the hearing aid periodically.

### b) Water Prevention

Keep hearing aid away from water. Please take off the hearing aid in advance and keep properly when swimming, shower, washing the head or face. If hearing aid falls into the water accidentally, do not use any dryer to dry it. You can use the soft and clean cotton cloth to dry it and put it in a ventilated and dry environment. If any failure occurs, please contact the hearing center where you acquired the hearing aid or customer service center of Jordan TE.CH. SA company.

### c) High Temperature Prevention

Never expose the hearing aid to high-temperature or prolonged sunlight.

#### d) Dropping Prevention

Hearing aid is precise electronic device. Please prevent it from dropping or violent vibration and shock.

EN

#### e) Others

- Do not wear your hearing aids during short-wave diathermy, X-ray, MR scans, CT scans or similar radiation treatments.
- Please use professional battery for hearing aid.
- If you not use hearing aid for long time, please take out the battery to avoid battery leakage.
- Battery should be put in a safe place which is out of baby or children's reach. If any swallowing accident happens, please consult a doctor immediately.

## Hearing Rehabilitation Plan

Every person reacts differently the first time he or she wears hearing aids. The familiarization phase therefore differs from person to person. Below is a rehabilitation plan for first time users or those using new hearing aids.

### a) The First Stage (1-3 weeks)

1 to 2 hours wearing per day, you are suggested to wear in a relatively quiet environment to familiarize and distinguish various sounds.

### b) The Second Stage (1-3 weeks)

4 to 8 hours wearing per day, you may wear your hearing aids and walk in a relatively quiet environment to listen around.

**c) The Third Stage (1-3 weeks)**

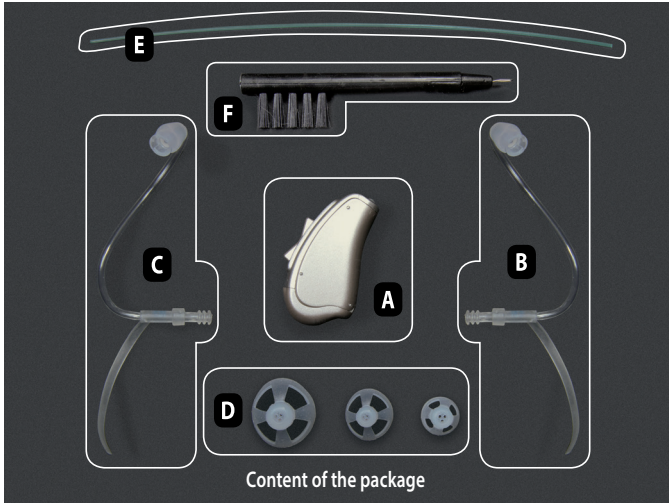
You may wear hearing aids and go to public areas. You may have dialogue with your friends or even strangers.

**d) The Fourth Stage (1-3 weeks)**

You may prepare for travelling to another city, go to supermarket, restaurant, participate in music concert or watch video.

EN

The unit can be worn on either ear, using the respective BTE tubes.

The kit consists of:



- A** Hearing Aid
- B** Right BTE tube
- C** Left BTE tube
- D** Earbuds in different sizes, S-M-L.
- E** Rigid wire for BTE tubes cleaning.
- F** Ear cleaning brush.

REF 151

EN

**BATTERIES  
NOT INCLUDED**

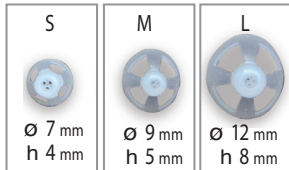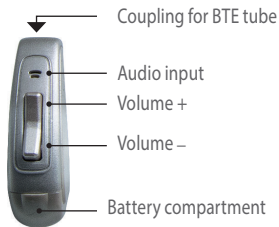
**Compatible Batteries:  
ZINC-AIR,  
model 312.**

Select the BTE tube: the right one, **RED B** or the left, **BLUE C**, depending on which ear the hearing aid will be applied on. Apply the chosen tube to the device: the end of the tube is to be tightened clockwise in the threaded plug, exerting pressure.

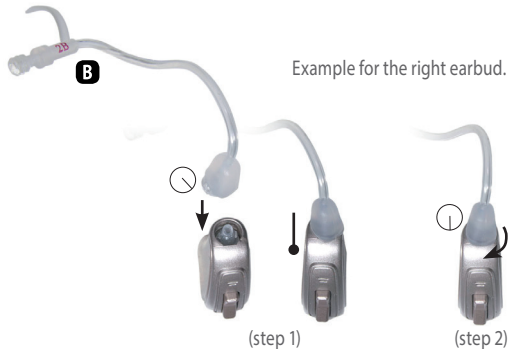
EN


Insert the end of the tube rotated by a quarter in the bayonet shaped plug (step1) and rotate it in the opposite direction to secure it (step 2). To remove the tube rotate in the reverse direction again.

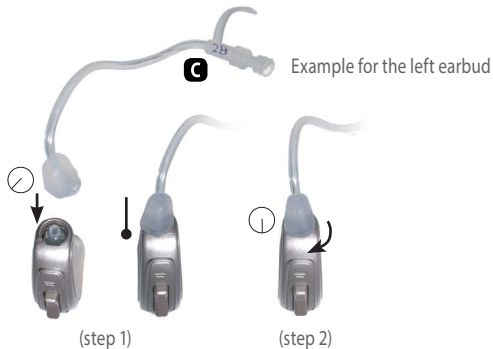EN


EN

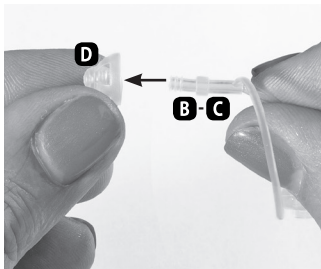Choose the most suitable earbud **D** between the three different sizes available (S-M-L). If this is your first time using the device, you may want to try M size earbuds first. Assemble by pressing the earbud **D** on the BTE tubes **B** or **C**.

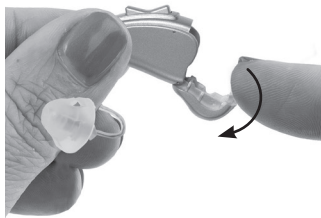
Open the battery compartment door. Do not push too hard, to avoid damage to the compartment.

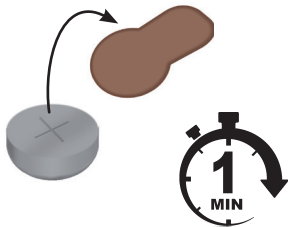
If you are using zinc-air batteries:  
remove the plastic battery and wait  
about 1 minute before inserting it back  
into the compartment.

**Compatible Batteries:**  
**ZINC-AIR, model 312**

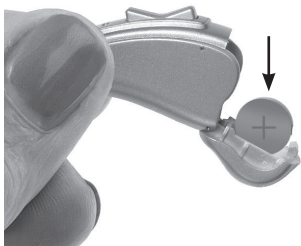
To turn o the unit open the battery  
compartment door.

**EN**

Close the battery compartment gently. If the battery is not properly positioned the battery compartment cannot be closed.  
The device is now switched on.

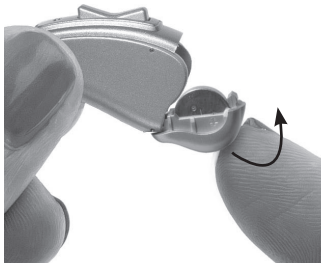EN

Upon first use, turn the volume to a minimum by pushing the volume button (see figure): Press the bottom to decrease it. Verify the functioning of the device by placing it between two hands closed: if the device is working properly it will produce a sound/whistle (Larsen effect).

EN

Press to  
lower the volume


### Installing the hearing aid:

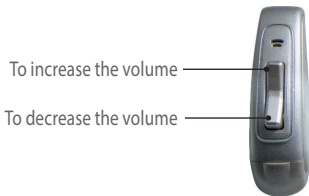1. Make sure you have properly inserted the battery and you have closed the compartment.
2. Pull up on the top of your ear to help the earbuds introduction into the ear canal and the flap of the tube in the ear.
3. Hold the headset with your thumb and index finger on the outside, next to the tube.
4. Gently insert, slightly tilted, the tip of the earbud in the ear canal.
5. Turn hearing aid rearwards.
6. Gently push the earbud with the fingertips until it is properly seated.
7. Place the hearing aid behind the ear.

You may hear a slightly annoying sound that will disappear when the device is properly positioned within the ear canal. If the positioning is correct and the sound is still perceptible, try with a different size and repeat the operation. Slowly adjust the volume as needed. The Hearing aid has seven different volume intensities marked each one by a "beep". The end of the scale, both for minimum and maximum volume, is marked by two "beeps"

EN

The prominence highlighted with the arrow is the stabilizer of the tube and must be inserted into the earcup.

EN

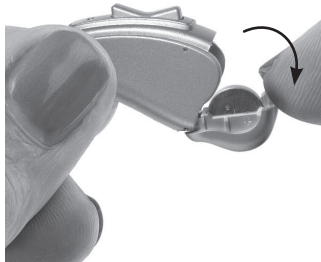
Once the headset is inserted into the ear canal, place the stabilizer inside the earcup to complete the assembly.

At the end of use, remove the device.

To turn off the unit open the battery compartment door.

EN

## Maintenance

When using hearing aids, the earwax accumulates in the ear canal and in the tube. The ear wax can affect the quality of the hearing aid. Please clean the ear canal and the hearing aid regularly.

EN

### a) Change the earbuds

- Remove the earbuds from the hearing aid  
Hold the device with your left hand and the tube with your right hand to separate them from each other.
- Install the earbud

### b) Replace the tube

- Remove the tube  
Hold the triangular base of the tube and turn it counterclockwise approximately 45 °.  
Remove the tube from the device.
- Assemble the new tube.

c) Clean the hearing aid

- \* Clean the hearing aid with a soft, dry cloth.

- \* Clean the hearing aid on a soft ground like the desk with a soft towel to avoid damage when falling down.

EN

**Warning:** never use any liquid to clean the hearing aid

d) Humidity prevention

Place the hearing aid in the box to prevent moisture and prolong the hearing aid.

e) Impact Prevention

Please place the hearing aid in the box or when not in use.

EN

# ((( DIGITAL INVISIBLE · 3D

REF 102

Other codes:

## Digital Micro 3D

REF 105 and 106

## Color Digital Micro 3D

REF 107 and 108

## Color Digital Invisible 3D

REF 109 and 110

REF 103

## Hearing aid description

The Digital Invisible 3D Polaroid hearing aid is digital, with 4 signal processing channels, intelligent and adaptive noise reduction system, highly effective acoustic feedback suppression function.

The Digital Invisible 3D Polaroid hearing aid helps the user not only to have a clean hearing in a quiet environment, but also in a noisy environment.

The product design meets the ergonomics with comfortable and powerful functions.

The description is the same for the **Digital Micro 3D, Color Digital Micro 3D, Color Digital Invisible 3D Polaroid** Hearing Aid.

EN

**EN**

Technical Specification	DIGITAL INVISIBLE-3D
	Values
MAX-OSPL 90	111.9 dB
HFA OSPL 90	107.9 dB
Peak Gain	37.2 dB
HFA Full on Gain	33.1 dB
Total Harmonic Distortion:	
500Hz (%)	0.5
800Hz (%)	0.2
1600Hz (%)	0.5
EQ Input Noise Level	22.9 dB
Frequency Range	$\leq 200 \text{ Hz} \geq 8000 \text{ Hz}$
Attack Time	46 ms
Release Time	54 ms
Battery Current	1.69 mA

## Structure Characteristics and Operating Principle

Hearing aid is consisted by the microphone, integrated circuit and receiver. The microphone can transform the received sound to electric signal, the integrated circuit will amplify the electric signal, the receiver will accept the amplified electric signal and restore to sound, to achieve the sound amplification purpose and make the compensation for the hearing loss user, to help the hearing loss user working and living like normal people.

EN

### Operating Condition

Input power: 3,5 mW

Atmospheric pressure: 86kPa~106kPA

#### Working Conditions:

Temperature: 0°C~+38°C, Humidity: 0%~80%, Voltage: DC1.4V

#### Storage Conditions:

Temperature: -20°C~+50°C, Humidity: 0%~93%

### Impact on the Environment

Please dispose of the used battery according to local regulations to protect the environment.

## Safety

EN

- **Warning:** Keep small parts that can be swallowed out of children's reach.
- **Warning:** Hearing aids are able to provide more than 132 dB SPL.  
We suggest having the acoustic amplifier mounted by a professional operator
- **Warning:** Not to use the hearing aids in such areas with risk of explosive or oxygen enriched atmospheres.
- **Warning:** The hearing aid must only be used by tended person and not by others.  
For hearing aids with wireless transmission: warning to check first before using the hearing aids in areas where electronics or wireless devices are restricted;  
Statement required about the special needs of particular users.  
e.g. small children or mentally disabled persons.
- **Warning** about common conditions that:  
**Danger:** Take out battery if there is long time non-use of hearing aids.  
**Danger:** Do not charge the battery or throw the used batteries into fire to avoid any explosion.

**Danger:** Do not mix the battery with food, medicines; Do not put the battery into the mouth. If a battery is swallowed, please consult a doctor immediately. It is life-threatening to swallow the hearing aid battery.

The hearing aid should be only used by the hearing impaired, not by any other.

- **Special User:**

- (1) Children can't wear the hearing aid without the guidance of adult.
- (2) Mental patients should consult a physician before wearing a hearing aid.
- (3) The hearing aid is not applicable for children under age of 3 years.

EN

## Simple Failure Analysis and Troubleshooting

EN

FAILURE	CAUSE	SOLUTION
Whistling	Hearing aid is not worn correctly	Correct and wear it again
	The volume is too loud	Turn down the volume
	Cracked hearing aid	Contact sales support
	Your hands or other objects are very close to the hearing aid	Move your hands or other objects away from ears or position the objects at a sufficient distance from the ear
	Too much earwax	Clean earwax or sales support
Silence	Power off	Turn on
	Hearing aid battery is not installed	Install the battery
	Low battery or no power	Replace it with new battery
	Din blocks the earplugor tube	Clean earwax or sales support
Small voice Not clear Distortion	Low battery	Replace it with new battery
	Hearing loss change	Consult the audiologist
	Hearing aid is damped	Dry with a cloth or sales support
	Din blocks the sound tube	Clean the sound tube or sales support

FAILURE	CAUSE	SOLUTION
Cant adjust the volume	Volume has been placed on the maximum or minimum level	Rotate the volume knob in the opposite direction
	Dirt blocks the sound tube or earplug	Clean it or sales support
High power consumption	Hearing aid is still on when not in use	Please turn it off when hearing aid is not in use
	The battery is expired	Check the validity of the battery
	Hearing aid is damped	Dry it with a cloth or sales support
	Poor battery quality	Replace it with new battery
	Local leakage	Call our sales support
Intermittent sound	Battery failure	Replace it with new battery
	Poor contact between battery	Call sales support

EN

If the problem is not solved by applying the procedures above, please contact the store where you purchased it or contact Jordan TE.CH customer service

## Maintenance and Repair

When use hearing aids, earwax will accumulate in the ear canal and ear plugs loud mouth. Large accumulation of eawwax may affect the sound quality of hearing aids, please regularly clean your ear canal and healing aids earplug.

EN

### Cleaning hearing aid

Clean your hearing aid with a dry soft cloth. Use a small brush and a soft dry cloth to clean the earwax in the hearing aid sound hole.

Try to perform all cleaning above a soft cloth or towel on a desk or table to keep the hearing aids from being damaged when falling down.

Never use any cleaning solution or other liquid like water to clean it since they may lead to pemwanent damage to the circuitry.

## Precautions for hearing aid

- a) **Moisture proof:** Moisture-proof is the core issue of hearing aid and is the key to the normal use and life extension of hearing aids. Place it in a "Dri-inshument Kit" to draw damaging moisture out of the instmments overnight, to extend the life of the hearing aid.
- b) **Waterproof:** Please do not wear the hearing aid when swimming. Remove the hearing aid when it is raining. If hearing aid accidentally falls into the water, do not use dryer to dry it. You can dry it with a soft, clean cotton cloth and put it in a ventilated environment.
- c) **Anti-high temperature:** Never expose it to extreme temperatures or prolonged exposure to sunlight.
- d) **Anti-vibration:** Do not drop your hearing aids or Knock them against hard surfaces.
- e) **Others** The products maximum output sound pressure level is under the product software control, which ensures the end user could not be exposed at the sound pressure level larger than 132 dB, no matter in daily use or single failure.

EN

- Do not wear your hearing aids during X-ray, MR scans, CT scans, short-wave diathermy, or similar radiation treatments.
- Please take off your hearing aids before you go into high electrical field or strong magnetic field and leave your hearing aids outside. Such as transformer substation, power station, labs etc.
- EMC: Our hearing aids meet bystander compatibility, but don't meet user compatibility. Please take off your hearing aids before you get close to radio and telecommunication equipments, otherwise that could damage your hearing or hearing aids.
- Please use special batteries for hearing aids.
- Should you not use them for a day or more, take out the battery to avoid battery leakage.
- Waste batteries should be out of reach from infants or children to avoid being swallowed. Once find that children swallow the battery, please immediately send him/her to hospital.

## Hearing Rehabilitation Plan

Every person reacts differently the first time he or she wears hearing aids. The familiarization phase therefore differs from person to person. Below is a rehabilitation plan for first time users or those using new hearing aids.

EN

### a) The First Stage (1-3 weeks)

1 to 2 hours wearing per day, you are suggested to wear in a relatively quiet environment to familiarize and distinguish various sounds.

### b) The Second Stage (1-3 weeks)

4 to 8 hours wearing per day, you may wear your hearing aids and walk in a relatively quiet environment to listen around.

### c) The Third Stage (1-3 weeks)

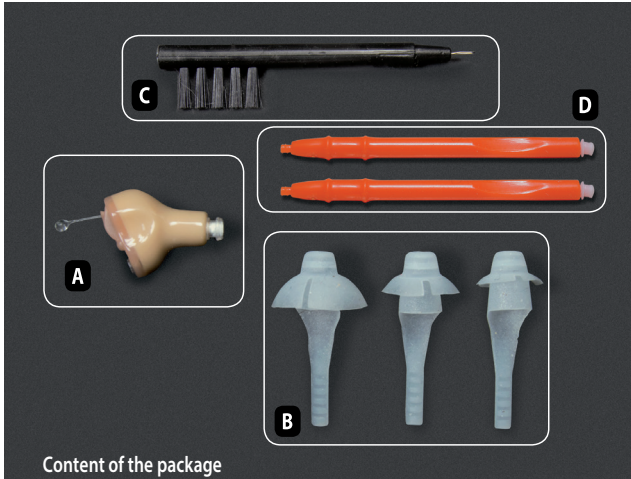
You may wear hearing aids and go to public areas. You may have dialogue with your friends or even strangers.

### d) The Fourth Stage (1-3 weeks)

You may prepare for travelling to another city, go to supermarket, restaurant, participate in music concert or watch video.

EN

Content of the package

Model with text in **RED** for the **right** ear.  
Model with text in **BLUE** for the **left** ear.

The kit consists of:



- A** Hearing Aid.
- B** Earbuds in different sizes, S-M-L.
- C** Ear cleaning brush.
- D** Wax protector and the extractor (2 pcs).

REF 152


**BATTERIES  
NOT INCLUDED**

**Compatible Batteries:  
ZINC-AIR, model 10.**

**EN**

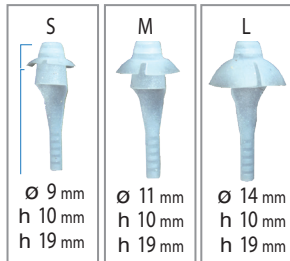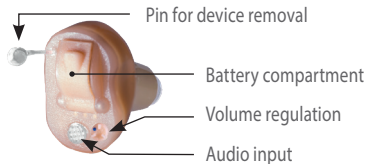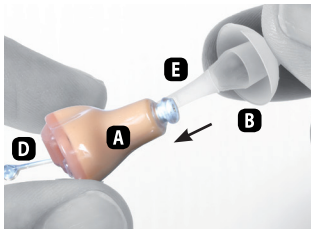
Choose the most suitable earbud between the three different sizes available (S-M-L) **B** and connect it to the speaker **A**, ensuring that it fits tight. The removing pin **D** must be in correspondence with the earbud removal pin **E**: this will facilitate the removal of the device after use. The first time you use the device you may want to try M size first.

Open the battery compartment. Do not push hard, so as not to damage the compartment.

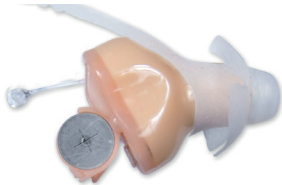
Insert the battery into the compartment ensuring correct polarity.

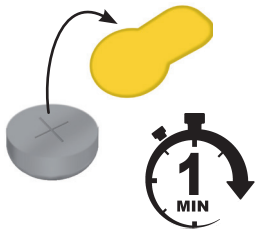Verify that the battery model is correct then remove the plastic battery and wait about 1 minute before inserting it back into the compartment.

**Compatible Batteries:**  
**ZINC-AIR, Model 10.**

Close the battery compartment gently. If the battery is not properly positioned the battery compartment cannot be closed. The device is now switched on.



**EN**

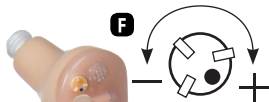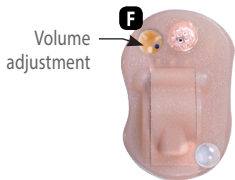
Before use, turn the volume to a minimum. To adjust the volume use your finger to rotate the small wheel: rotate counterclockwise to decrease the volume, clockwise to increase it **F**. Verify the functioning of the device by placing it between two hands closed: if the device is working properly it will produce a sound/whistle (Larsen effect).

**EN**

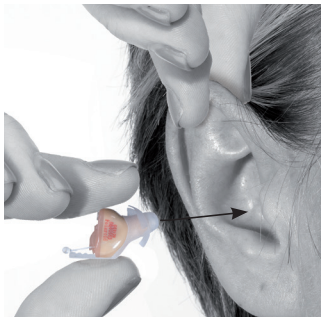
Be sure to check that the **right** earbud (**red**) and the **left** earbud (**blue**) are applied each one to the respective ear. Pull up on the top of the ear and insert the device into the ear canal.

You may hear a slightly annoying sound that will disappear when the device is properly positioned within the ear canal. If the positioning is correct and the sound is still perceptible, try with a different size and repeat the operation.

**EN**

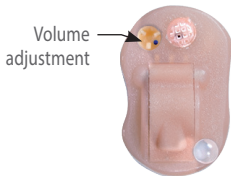Slowly adjust the volume as needed.

At the end of use, remove the device.

EN

To turn off the unit, open the battery compartment door.

## Maintenance

When using hearing aids, the ear wax accumulates in the ear canal and in wax protector. The ear wax can affect the quality of the hearing aid. Please clean the ear canal and replace the wax protector regularly.

EN

### a) Change the earbuds

- Remove the earbuds from the hearing aid  
Hold the device with your left hand and the earbud with your right hand to separate them from each other.
- Install the earbud

### b) Replace the wax protector

- See the instructions in the following pages

The wax protector is used to protect your hearing aid from any occlusion of the sound exit hole due to the wax.

**EN**

To replace the wax protector, use the specific extractor. The extractor has two sides: one free and the other with the new filter already inserted.

To remove the dirty filter, use the free tip of the extractor, simply inserting it into the wax protector and pulling it outward.

Now use the other side of the extractor to insert the new wax protector into the newly released cavity.

EN

Then gently remove the extractor.

c) Clean the hearing aid

\* Clean the hearing aid with a soft, dry cloth.

\* Clean the hearing aid on a soft ground like the desk with a soft towel to avoid damage when falling down.

EN

**Warning:** Never use any liquid to clean the hearing aid

d) Humidity prevention

Place the hearing aid in the box to prevent moisture and prolong the hearing aid.

e) Impact Prevention

Please place the hearing aid in the box or when not in use.

## SYMBOL INFORMATION

Manufacturer

Complies with MDD 93/42 / EEC requirements

Protect from humidity

Keep away from sunlight

Do not use if package is damaged

Caution, read instructions for use

Fragile, handle with care

Read instructions for use

Waste EEE

Recycled or recyclable packaging

Date of manufacture

Serial number

Batch number

Catalog number

EN

Jordan TE.CH. SA - via Mondadiscio 20 - Agno, Switzerland

©2018 All Rights Reserved. Polaroid, Polaroid & Pixel, Polaroid Color Spectrum and Polaroid Classic Border Logo are trademarks of PLR IP Holdings, LLC, used under license. Trademarks of PLR IP Holdings, LLC does not manufacture this product or provide any Manufacturer's Warranty or support.

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Instantly recognizable. Instantly reassuring. The Polaroid Classic Border and Polaroid Color Spectrum logos let you know you've purchased a product that exemplifies the best qualities of our brand and that contribute to our rich heritage of quality and innovation.